OCULAR INSTRUMENTS

2255 116th Ave NE, Bellevue, Washington 98004-3039 USA

T: 425-455-5200 or 800-888-6616 F: 425-462-6669

E: contact@ocularinc.com I: www.ocularinc.com

EC	REP	EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands
---------------	----------------	--

Product Care Instructions - english

CLEANING METHOD 2

For All:

Ocular MaxField® Glass and MaxLight® CR-39 Indirect Diagnostic/Laser Lenses EXCEPT: (See cleaning Method 3) OI-20A, MaxAC 20D Indirect and OI-28D, MaxAC 28D Indirect.

Cleaning

Wipe: Clean with alcohol wipe.

Then: Proceed with either disinfection or sterilization instructions.

Disinfection

SOAK IN: Glutaraldehyde - 2% or 3.4% aqueous solution, Temperature per manufacturer instructions, Minimum exposure time = 20 minutes OR **BLEACH** 10% solution mixed at: 1 part bleach to 9 parts cool or tepid water. Recommended exposure time = 10 minutes. **Caution** - *To avoid damage to the lens, do not exceed recommended exposure time.*

Then: Rinse lens *thoroughly* to remove disinfection solution.3 cycles of 1 minute, with cool or tepid water is recommended. Dry carefully and place in a dry storage case.

NOTE: These lenses are known to be compatible with: Asepti-Wipe, Cavi-cide, Cidex, Cidex OPA, DisCide Wipe, Tristel Wipes System, Enviro-cide, H₂O₂-3% and Opti-Cide. MaxField® lenses are also compatible with Steris Resert except OI-120M, OI-60M, OI-66M, OI-HM-78M, OI-78M, OI-SP.

Sterilization

AUTOClave – No, STERRAD – NO, STERIS SYSTEM 1 – Yes, Follow manufacturer instructions for MaxField® (glass) lenses. Not compatible with MaxLight® (CR-39) lenses.

ETO – Yes, EO Parameters - Minimum Time: 1 hour, Temperature: 130°F (54°C), Aeration Time: 12 hours.

WARNING - Never Steam Autoclave or Boil listed lenses. Never soak in Alcohol, H₂O₂, Acetone or Other Solvents.

For information on compatibility with alternative product care methods, contact Customer Service.

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