



Instructions for Ophthalmoscopes

Repairs

Repairs should be performed at the factory.

Cleaning and Disinfection

For cleaning the mirror or lenses, use only a clean, dry optical-grade cloth. Do not use alcohol or other chemical solutions. The window may be cleaned with a cotton swab or lens paper dipped in alcohol.

The Head housing may be cleaned and disinfected with an appropriate health care low- or intermediate-level cleaner/disinfecting wipe that incorporates either a 1:10 sodium hypochlorite (bleach) solution or isopropyl alcohol as the active disinfection ingredient. Follow wipe manufacturer's instructions for appropriate use, contact times and applicable warnings and precautions.

Do not excessively saturate the Head.

Do not immerse Head in any solution.

Do not sterilize the Head.

After disinfection, inspect for visible signs of deterioration. If evidence of damage or deterioration is present, discontinue use and contact Welch Allyn or your area representative.

Instructions for Otoscopes

Repairs

Repairs should be performed at the factory.

Cleaning and Disinfection

For cleaning the mirror or lenses, use only a clean, dry optical-grade cloth. Do not use alcohol or other chemical solutions.

The window may be cleaned with a cotton swab or lens paper dipped in alcohol.

The Head housing may be cleaned and disinfected with an appropriate health care low- or intermediate-level cleaner/disinfecting wipe that incorporates either a 1:10 sodium hypochlorite (bleach) solution or isopropyl alcohol as the active disinfection ingredient.

Follow wipe manufacturer's instructions for appropriate use, contact times and applicable warnings and precautions.

Do not excessively saturate the Head.

Do not immerse Head in any solution.

Do not sterilize the Head. After disinfection, inspect for visible signs of deterioration. If evidence of damage or deterioration is present, discontinue use and contact Welch Allyn or your area representative.



Hillrom.

4341 State Street Road
Skaneateles Falls, NY 13153

1 800 535 6663

hillrom.com